

## CD99 / MIC2

## IN VITRO DIAGNOSTIC DATASHEET

**INTENDED USE :** IN VITRO DIAGNOSTIC USE

This product is intended for qualitative immunohistochemistry with normal and neoplastic formalin-fixed, paraffin-embedded tissue sections, to be viewed by light microscopy.

**DESCRIPTION :**

CD99, or MIC2 gene product, or E2 antigen is expressed on the cell membrane of some lymphocytes, cortical thymocytes, and granulosa cells of the ovary. The antigen is also expressed by most pancreatic islet cells, Sertoli cells of the testis, and some endothelial cells. Mature granulocytes express very little or no CD99.

<b>CATALOG NO :</b>	PL294	PL294-R7	7 ML RTU 70 TEST
		PL294-R1	1 ML RTU 10 TEST
<b>STAINING PATTERN :</b>	Cell membrane	PL294-1	1 ML 1/100 1000 TEST
		PL294-0,1	0,1 ML 1/100 100 TEST
<b>POSITIVE CONTROL :</b>	Pancreas or Ewing's sarcoma		

**VOLUME :** 7 ml Ready to Use ( 7 ml of antibody prediluted in 0.05mol/L Tris-HCl, pH 7.6 containing stabilizing protein and 0.015mol/L sodium azide. )

**HOST :** Mouse

**CLONE :** HO36-1.1

**ANTIBODY CONCENTRATION :** 200ug / ml

**SPECIES REACTIVITY :** Human, Others not tested.

**EPITOPE :** Belirlenmedi.

**MICROBIOLOGICAL STATE :** This product is not sterile.

**PRETREATMENT :** Staining of formalin-fixed tissue sections requires treating the tissue sections in boiling 10mM citrate buffer, pH 6.0, for 10-20 minutes followed by cooling at room temperature for 20 min.

**PRIMARY ANTIBODY INCUBATION TIME :** 30 minutes at Room Temperature

**STAINING TIPS :** If the staining is too light, use lower dilution or longer time. If the staining is too strong, check pretreatment, use higher dilution or shorter time.

**STORAGE AND STABILITY :** This product contains sodium azide and is stable for 24 months when stored at 2-8°C. Do not use after expiration date indicated on label of the product. If reagent is not stored as recommended, performance must be validated by the user.

**TROUBLESHOOTING :** Please contact Patolab Technical Support by e-mail ( patolab@patolab.com.tr ).